





Inserting the cartridge

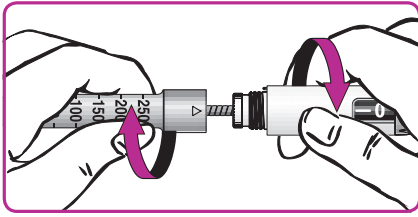


Please check that you have the right insulin first.


- A** Unscrew the cartridge sleeve from the pen housing
If necessary, remove the empty insulin cartridge.

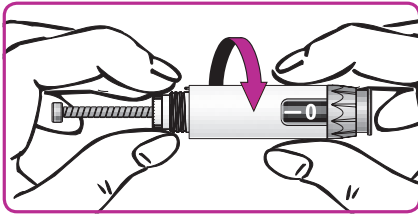


- B** Twist the threaded bar completely back into the start position.
- ▶ Hold the threaded bar ring firmly and turn the housing back as far as it will go.
 - ▶ **DO NOT TRY** to push the threaded bar back.

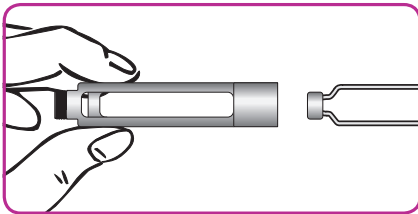

- C** Insert a new cartridge.
- ▶ Make sure that you are using the right insulin, and then insert the cartridge.

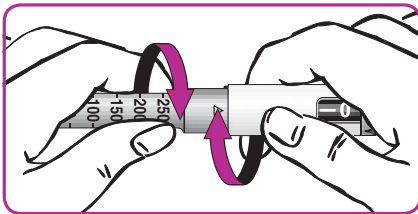

- D** Screw the cartridge sleeve back into the pen housing.

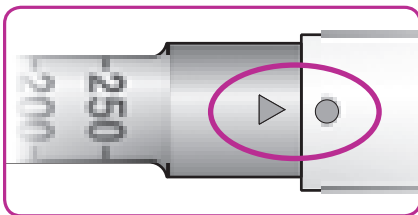

The markings must line up.

If this is not the case, start again at Step A.

If you cannot insert the cartridge, check that

- ▶ you have put the cartridge in the right way round
- ▶ the threaded bar has been fully turned back.

If you have other problems with the pen, try changing the needle and repeating the safety test.

The TactiPen® is designed solely for personal use.

You will find instructions for keeping and handling the cartridges in the pack insert enclosed with the insulin cartridges.


Always keep an alternative way of injecting your insulin as a back-up in case your TactiPen® is lost or damaged.

Use a new needle for each injection.

Injecting the insulin

IMPORTANT: Mix insulin suspensions!


1 ▶ Attach the **needle**

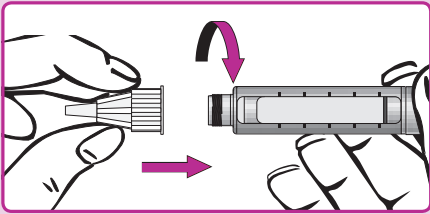

Please check that you have the right insulin first.

- A** Make sure the needle is straight when you attach it.


2 ▶ Carry out the **safety test**

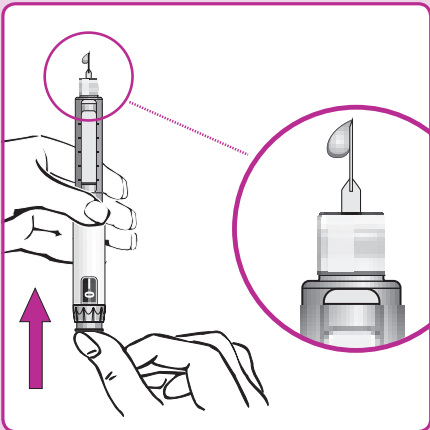

Before every injection, you should carry out a safety test to remove air bubbles and to check that the pen and needle are working properly.

- A** Set the dose at 2 units and remove the needle protection cap.
- B** Hold the pen with the needle upward and push the injection button in completely.
- C** Check that insulin comes out of the tip of the needle.
 - ▶ If no insulin comes out, repeat the test until insulin comes out.
 - ▶ If no insulin comes out after you have carried out the test three times, change the needle and try again.
 - ▶ If insulin still does not come out, make sure that the cartridge is correctly inserted and try again.
 - ▶ If insulin still does not come out, change the cartridge and start again.

3 ▶ Set the **dose**

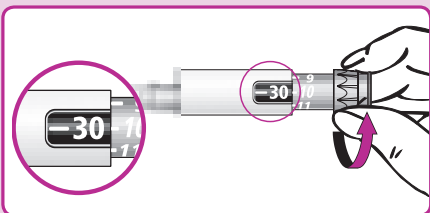


- A** Turn the injection button to set the dose.

In the example shown, a dose of 30 units has been selected. Your dose may be different.


IF YOU cannot set the required dose, check that there is enough insulin in the cartridge.

4 ▶ Inject the **dose**

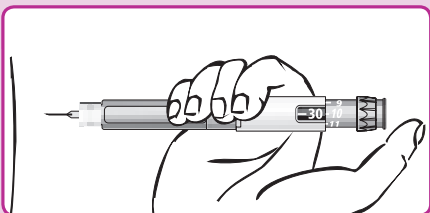


Use the injection technique that you have been shown by the medical specialists.

- A** Insert the needle

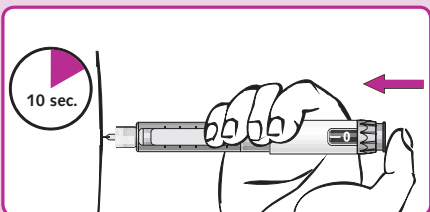

- B** Press the injection button in completely. The figure shown in the dose window returns to „0“.
- C** Keep the injection button pressed in completely. Count slowly to 10 before removing the needle.

5 ▶ Remove the **needle**

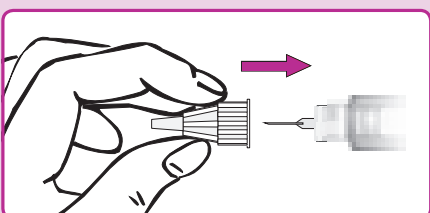

- A** Replace the outer needle protection cap on the needle and use it to twist the needle off the pen.
- B** Dispose of the needle carefully (e.g. in a puncture-proof container).
- C** Replace the pen cap on the pen.